




DECUBITUS THERAPY SEAT CUSHION

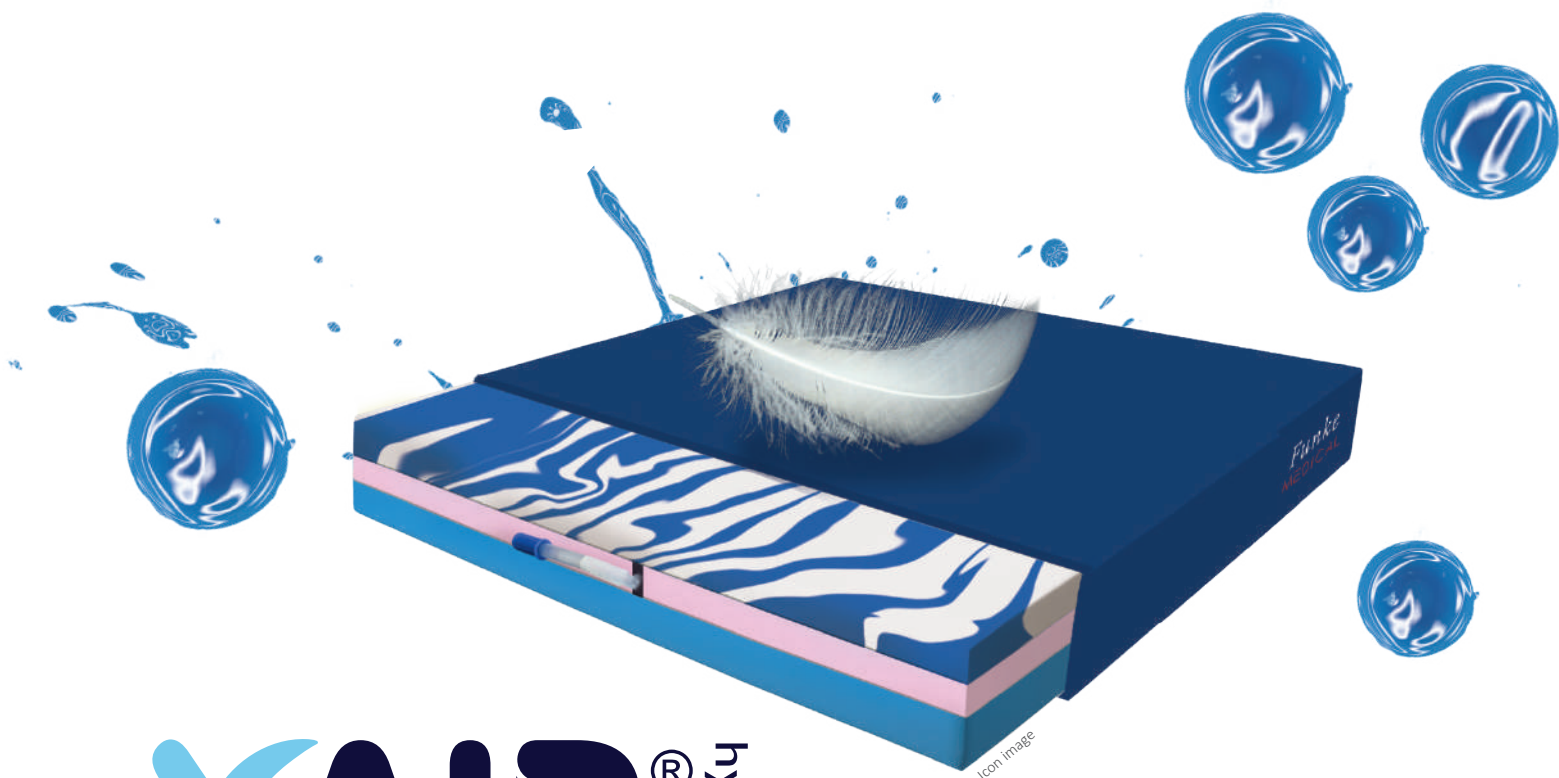

XAIR® hybrid

- AIR-CELL SEAT CUSHION -

For active wheelchair users!

Your retailer:

Official sponsor



2010: Silver World Championship (Birmingham, GB)
2011: Gold European Championship (Nazareth, IL)
2012: Gold Summer-Paralympics (London, GB)
2013: Silver European Championship (Frankfurt, DE)
2014: Silver World Championship (Toronto, CA)
2015: Gold European Championship (Worcester, GB)
2016: Silver Summer-Paralympics (Rio de Janeiro, BR)
2017: Silver European Championship (Tenerife, ES)
2018: Bronze World Championship (Hamburg, DE)
2019: Bronze European Championship (Rotterdam, NL)

A Breuer
Annabel Breuer
National player
-Wheelchair Basketball-



Autoclavable according to
RKI*¹ guidelines


Applicable up to stage 4
(EPUAP)

Very high risk
(Norton-, Braden-, Waterlow-Scale)

XAIR[®] hybrid

Decubitus therapy air-cell seat cushion
for active wheelchair users!

3 YEARS
GUARANTEE
*²

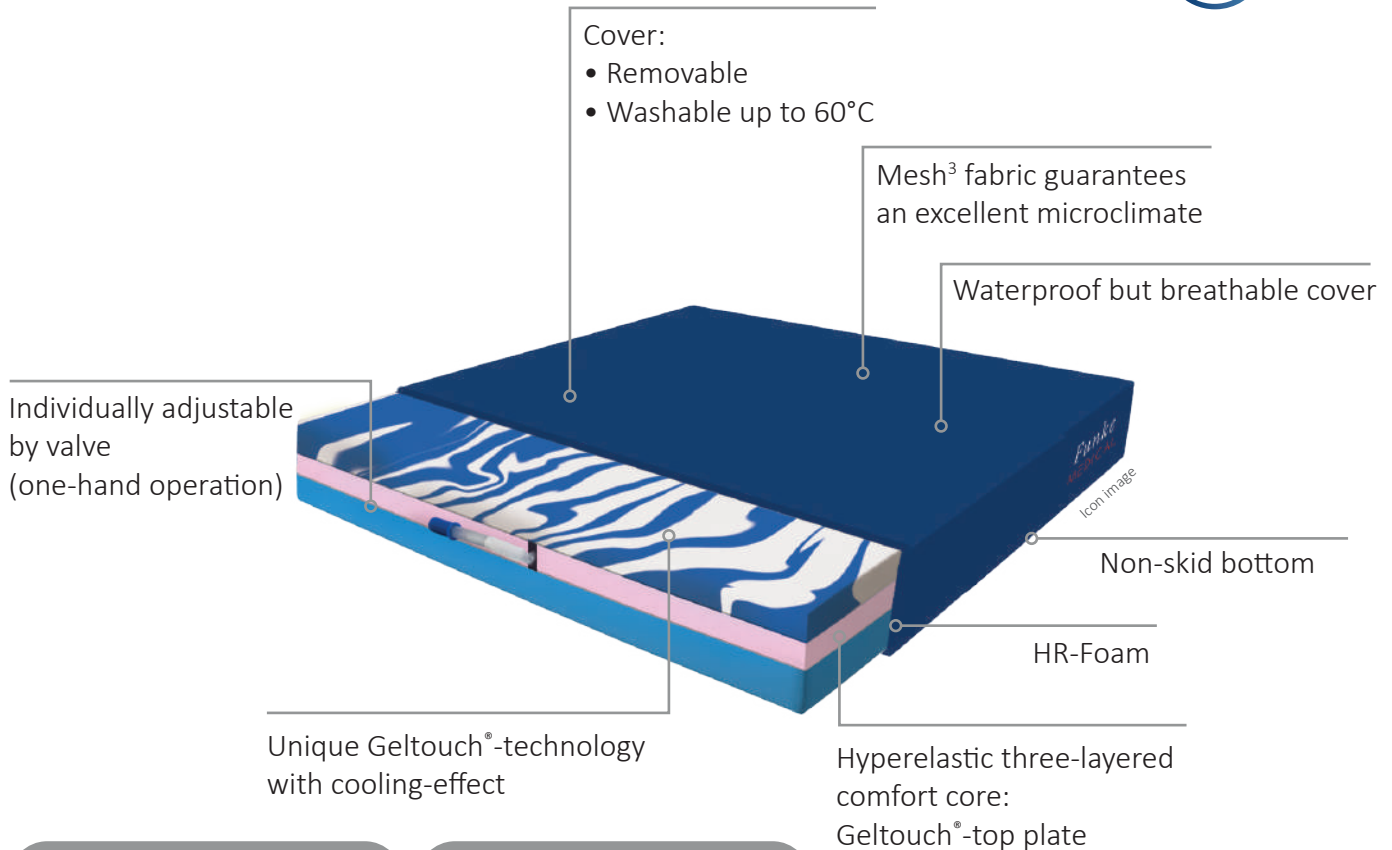


Size in cm	Item No.
Width x Length x Height	
38 x 40 x 8	XAIR3840
40 x 40 x 8	XAIR4040
40 x 42 x 8	XAIR4042
41 x 46 x 8	XAIR4146
42 x 40 x 8	XAIR4240
42 x 42 x 8	XAIR4242
42 x 44 x 8	XAIR4244
42 x 46 x 8	XAIR4246
42 x 52 x 8	XAIR4252
45 x 42 x 8	XAIR4542

Size in cm	Item No.
Width x Length x Height	
45 x 44 x 8	XAIR4544
45 x 46 x 8	XAIR4546
46 x 51 x 8	XAIR4651
48 x 42 x 8	XAIR4842
48 x 44 x 8	XAIR4844
48 x 46 x 8	XAIR4846
50 x 42 x 8	XAIR5042
51 x 44 x 8	XAIR5144
51 x 46 x 8	XAIR5146
52 x 52 x 8	XAIR5252

WATCH NOW
on YouTube!



XAIR[®] hybrid-Movie

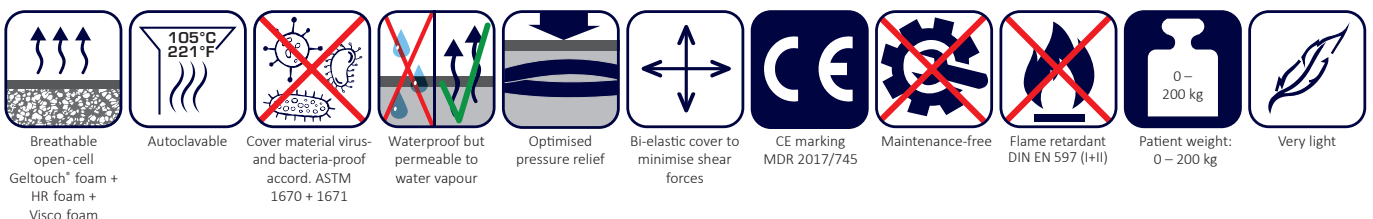

*¹ Robert-Koch-Institute (RKI) is a part of the Federal Ministry of Health in Germany. It is responsible for disease prevention and control.

*² Guarantee applies to functional properties of the cushion core (pro rata basis)!



WATCH NOW
on YouTube!




Handle for easy carrying

Soft protective cover with
hook-and-loop fastener




1

Lift the protective cover.

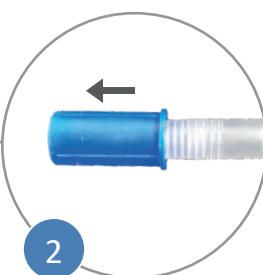

2

Open the valve by pulling the blue
lock and wait approx. 30 seconds,
until the air-cells have inflated
automatically.
(Valve lock can not fall off.)

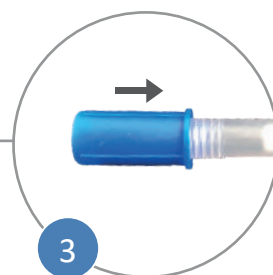

3

Close the valve
by pressing.




4

Put the cushion on the wheelchair or seat.


(The handle must point to the front and the
attached arrows must point upwards.)

Sit on the cushion.

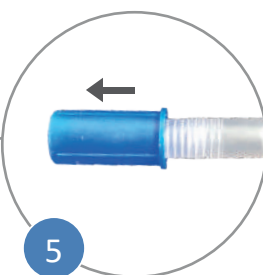



5

Carefully open the valve again
until the ideal sitting comfort
is achieved.

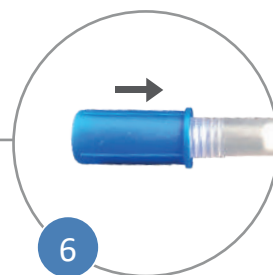

6

Close the valve and
the protective cover.

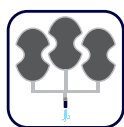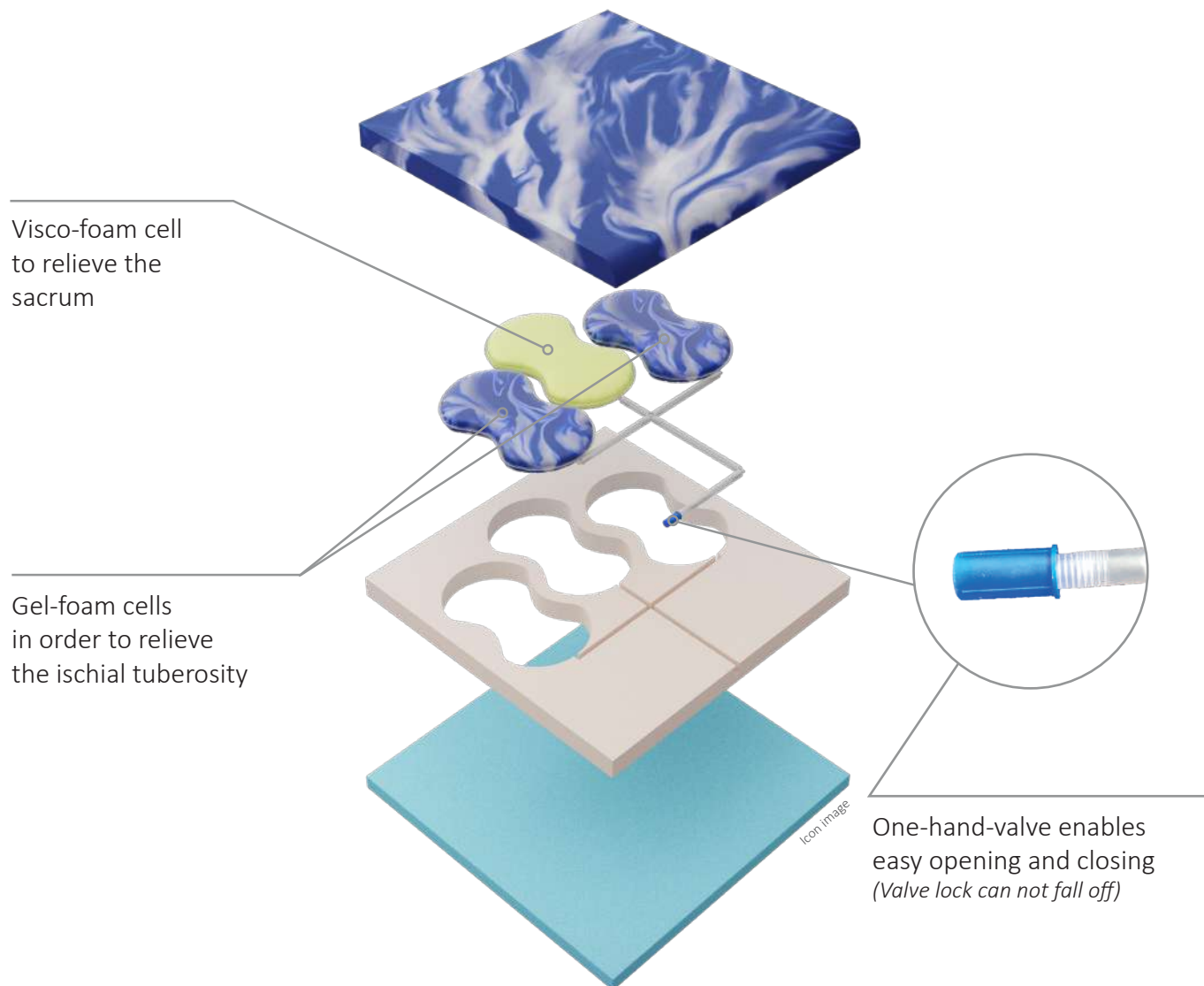

3-chamber seat cushion provides optimum pressure relief



Individually adjustable for ideal seating comfort



No pump or other tools necessary



Fast and easy adaptation to external pressure conditions



No risk of failure due to self-inflating FOAM-IN-CELL-TECHNOLOGY



Very light – 70 % less weight than conventional air-cell seat cushions